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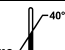
Ed. 01. 18.05.2024
IFU/ZDKT/02

CRICOTHYROTOMY SET – PACRIC INSTRUCTIONS FOR USE

These instructions for use refer to product REF 30-0001

Pictograms:

	Catalogue number		Manufacturer		Free from phthalates (incl. DEHP)
	Batch code		Unique device identifier		Medical device
	Date of manufacture		Free from latex		Do not re-use
	Use-by date		Consult instructions for use		Keep dry
	CE marking with Notified Body no.		Keep away from sunlight		Temperature limit 5-40°C
	Sterilized using ethylene oxide, Single sterile barrier system		Do not use if packaging is damaged		

Description

Cricothyrotomy is an invasive technique used in the emergency conditions as a quick and immediate manner of restoring patency of the airways, which have been subjected to the airway obstruction at the height or above the rima glottidis. The laryngological procedure consists in an incision of the cricothyroid ligament, located between an inferior margin of lamina of the larynx thyroid cartilage and a superior margin of the arch of the larynx cricoid cartilage. Cricothyrotomy is a procedure of choice if there are the emergency indications in case of a lack of the possibilities of a ventilation or a conventional intubation of the patient, used when all other methods of the airway management failed to be effective. Cricothyrotomy set is a single-use device.

Kit components

Tracheostomy tube size 6.0; Bougie introducer; 10 ml syringe; Scalpel N°10; Strap for tracheostomy tube

Intendent use

Cricothyrotomy set - PACRIC is used to perform the cricotomy procedure– an invasive technique used in emergencies as a quick and ad hoc way to clear airways that have been blocked at or above the glottis gap.

Indications

Inability to ventilate and oxygenate the patient in life-threatening conditions.

Contraindications

No absolute contraindications (age under 12 years)

Users

Healthcare professionals.

Target group

Patients requiring mechanical ventilation (age over 12 years)

Instructions for use

Before use, ensure whether a set is complete. Cricothyrotomy should be performed by an anaesthesiologist familiar with the procedure.

Warning! During the whole procedure you should remain calm and try to oxygenate the patient.

1A. Impalpable cricothyroid membrane

- Stand on the left side of the patient
- Place a pillow under the patient's shoulders and position his head to extend neck.
- Using left hand to stabilize larynx, tense the place of incision between your fingers
- Make a 8-10cm vertical incision, caudad to cephalad
- Use blunt dissection with fingers of both hands to separate tissues until you identify the cricothyroid membrane

1B. Palpable cricothyroid membrane

- Stand on the left side of the patient
- Place a pillow under the patient's shoulders and position his head to extend neck.
- Identify the cricothyroid membrane with left hand

- Using left hand to stabilize larynx, make a transverse stab incision through cricothyroid membrane
- Turn blade through 90° (sharp edge caudally)
- Hold the scalpel with your left hand. Insert the bougie introducer to 10-15cm, caudad
- Slide the tracheostomy tube along the guide, confirm the correct placement of the tube and secure its position with the strap.

Precautions

- Under no circumstances the end of the tracheostomy tube should be occluded, for example with speaking valve, decannulation cap or a similar object.
- Protect the cuff from damage due to contact with sharp edges

- Avoid a contact with the electrosurgical electrodes or laser radiation, because PVC will emit the toxic fumes or will cause an ignition of the air enriched with oxygen.
- The medical device is not proper for the patients during the radiotherapy.
- Do not use during the Magnetic Resonance Imaging (MRI).
- Under no circumstances you should continue to use kit components if the tracheostomy tube or the cuff has been damaged (e.g. cracks on the surface, a leaky cuff) or when there is loosening of connections between the flange and the tube, or if the flange separates from the tube.

Warnings

Because of a risk of infection of airways with some hospital strains that are treatment resistant, the current approved medical procedures and techniques should be observed. The tube is exclusively for single use.

Complications

Observance of the current approved medical techniques and performing of a procedure by qualified medical personnel reduces down to a minimum possibility of complications e.g.: stenosis of larynx, trachea, bleeding or infection of a wound mediastinal emphysema, tracheal and esophageal fistula, occlusion of the tracheostomy tube, infection and decubitus ulcer within a windpipe, leak of air around the tracheostomy tube, infection of respiratory system, dyspnoea and respiratory effort, inflammation of tracheal and bronchial coats, haemorrhages from larynx.

Notes

The set should be stored in 5-40°C, in a dark, dry place. Do not use in if package is damaged. Do not use after the expiry date.

Disposal

After use, dispose of this device in accordance with applicable national regulations on medical waste disposal.

Responsibility – legal guidelines

Products manufactured by SUMI undergo physical, biological and chemical tests. This fact allows to use them safely and guarantees top quality. It is recommended to thoroughly acquaint the instruction for use prior to use of the cricothyrotomy set. SUMI shall not bear responsibility for consequences caused by using the cricothyrotomy set contrary to their intended use or instructions for use. In the event of a complaint or an occurrence of a medical incident, keep the medical device with its immediate packaging. Photographic documentation is also recommended. Every medical incident have to be reported to the manufacturer and National Competent Authority

SUMI sp. z o.o. sp. k.
Drobiarska 35, 05-070 Sulejówek, Poland
tel/fax (+48 22) 783 30 23 www.sumi.com.pl

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